



Product Information

www.miller-stephenson.com

FluoroExtreme™ Dental Lubricant Application Guide

Miller-Stephenson's FluoroExtreme™ Dental Lubricants are fluorinated synthetic lubricants formulated specifically for dental handpieces. These lubricants offer superior protection, lubricity, and chemical stability versus hydrocarbon alternatives. Miller-Stephenson's dental lubricants are 100% biocompatible (ISO 10993) and nonflammable. FluoroExtreme™ comes as an aerosol (MS-4130DA) or in a syringe-tipped dropper bottle (MS-4010DA).

Please follow the steps below for the cleaning and lubrication of your dental handpieces using FluoroExtreme™ Dental Lubricant:

Handpiece Preparation Prior to Initial Application of FluoroExtreme™:

1. Handpieces must be thoroughly purged of all previous lubricants prior to application of FluoroExtreme™ via utilization of milSolv™ Surface PrepClean™ MS-255DA. Miller-Stephenson cannot guarantee the performance of our lubricant system if this is not performed.
2. Insert MS-255DA nozzle into handpiece and spray until expelled fluid is clear.
3. Allow handpiece to dry completely before application of FluoroExtreme™.

Application of FluoroExtreme™ Dental Lubricant: MS-4130DA:

1. Fit aerosol can with appropriate nozzle tip. Shake can well, insert into handpiece and spray for 2 seconds with the head tilted down.
2. Run handpiece to expel any excess lubricant with the head tilted downward, cupping with a paper towel. Repeat steps 2-3 if expelled lubricant is discolored.
3. Wipe down handpiece and sterilize.

Application of FluoroExtreme™ Dental Lubricant: MS-4010DA:

1. Apply 2-3 drops of lubricant into the drive air port.
2. Run handpiece at normal speeds to expel any excess lubricant with the head tilted downward, cupping with a paper towel. Repeat steps 2-3 if expelled lubricant is discolored.
3. Wipe down handpiece and sterilize.

1580-7K

miller-stephenson logo, FluoroExtreme™, milSolv™ and PrepClean™ are trademarks of miller-stephenson chemical company inc.

For technical information call 800.992.2424 or 203.743.4447
For product sales: CT 800.442.3424, CA 800.771.8161, IL 800.447.4866, Canada 800.307.2199
www.miller-stephenson.com